DE

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EN

Cleaning and care of the material surface of Active Line artificial leather

Regular cleaning and care of the material surface is essential to maintain the material properties. We recommend using warm, mild soapy water and a microfibre cloth to do so. Please do not use chemical bleaches, acetone, petroleum ether, abrasive cleaning agents or other common household cleaners. Remove discolouration from foods or other substances promptly. In general, spots should always be treated immediately and not allowed to dry on.

Active Line artificial leather is highly resistant to soiling. Because of this characteristic, it is less susceptible to discolouration from denim than common artificial leathers.

In case of difficult stains, such as from permanent markers or pens, the surface of the material can also be cleaned with alcohol-based cleaners. Use isopropanol or methylated spirit for this purpose, for instance. Residues of alcohol-based cleaning agents and soap residue must then be removed using a moist cloth and lukewarm water.

To extend the life of your product, please clean and care for it regularly.

We recommend using the specially developed Active Line® Plus cleaner for artificial leather from LCK. Further product information is available at: www.activeline.moebelpflegeshop.de

Recommended disinfectants

To disinfect the surface of your product, use disinfectants listed in the overview below. Please note their reaction time and use them according to the manufacturer's information listed on the respective product.

Dr. Schumacher GmbH: Cleanisept / Wipes 5 %, Optisept 7 %, Biguanid Fläche N 3 %, Descosept PUR Wipes RTU, Descosept Sensitive Wipes, Descosept spezial Wipes, Ultrasol active 1 %

Bode Chemie GmbH: Microbac forte 3 %, Kohrsolin FF, Bacillol 30 foam, Dismozon plus 3,6 %

Schülke & Mayr GmbH: Perform 3 %, Mikrocid sensitive liquid undiluted, Mikrocid universal liquid undiluted, Terralin protect 2 %

Ecolab GmbH: Incidin liquid undiluted, Incidin Pro, Incidin Plus, Incidin Active 3 %, Incidin OxyWipe S

B. Braun Melsungen AG: Meliseptol foam pure undiluted, Meliseptol rapid undiluted

Dürr Dental AG: FD300 1 %, FD366 undiluted

Antiseptica Dr. H.-J. Molitor GmbH: Biguacid S 2 %, Descogen Liquid r.f.u.

Lysoform Dr. Hans Rosemann GmbH: Lysoformin spezial 0,75 %, Aldasan 2000 4 %

Tana Chemie GmbH: Apesin rapid 3 %

Aquagenius Schweiz GmbH: ASCEA-des undiluted

CosiMed GmbH: CosiMed Konzentrat 7,5 %

For full details regarding care and cleaning, as well as important product information for your upholstery fabric, please see www.hoepke.de/activeline or request information from your dealer.

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EN

Cleaning and care information Recommended disinfectants

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Bode Chemie GmbH: Microbac forte diluito al 3 %, Kohrsolin FF, Bacillol 30 foam, Dismozon plus diluito al 3,6 %
Ecolab GmbH: Incidin liquid non diluito, Incidin Pro, Incidin Plus, Incidin Active diluito al 3 %, Incidin OxyWipe S
B. Braun Melsungen AG: Meliseptol foam pure non diluito, Meliseptol rapid non diluito
Dürr Dental AG: FD300 diluito al 1 %, FD366 non diluito
Antiseptica Dr. H.-J. Molitor GmbH: Biguacid S diluito al 2 %, Descogen Liquid r.f.u.
Lysoform Dr. Hans Rosemann GmbH: Lysoformin spezial diluito al 0,75 %, Aldasan 2000 diluito al 4 %