Waterbased Silicone Fabrics

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Cleaning and care information /
Recommended disinfectants

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Regular cleaning and maintenance of the material surface are essential to preserve its properties.

For basic cleaning, it is sufficient to wipe the material surface with a damp, standard cotton cloth. For heavier soiling, we recommend using a mild soapy solution and a microfiber cloth.

Remove stains, soiling or other substances **immediately** and do not allow them to dry.

Always test cleaning products on a hidden area of the material beforehand.

Please avoid using chemical bleaches, acetone, cleaning benzene, abrasive cleaning agents, or other common household cleaners containing grease, oil, or solvent-based ingredients.

Color transfer from jeans or other non-colorfast materials is excluded from any warranty coverage.

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Our waterbased silicone fabrics are upholstery materials that stand out for their extreme resistance to external influences.

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For surface disinfection of your item, please use the disinfectants listed in the overview opposite.

Be sure to follow the exposure time and usage instructions provided by the manufacturer on the respective product. Höpke Textiles cannot assume any liability for changes in the formulation of disinfectants made by the manufacturers.

For matte surfaces, prolonged use of cleaning or disinfecting agents may cause the surface to appear slightly glossier. This does not constitute a material defect or impair the functional properties.

Testing Procedure: The disinfectants are applied in accordance with the manufacturer's instructions. Over a period of three months, the test samples are sprayed with the disinfectants three times daily. After the test period is complete, the test samples are evaluated visually and by touch.

MITTEL / PRODUCT	VERDÜNNUNG / DILUTION	RESULTAT / RESULT
Aldasan 2000	1,00 %	● geeignet / suitable
Antiseptica Kombi Liquid	unverdünnt / pure	● geeignet / suitable
Antifect N Liquid	unverdünnt / pure	● geeignet / suitable
Biguanid Flächen	2,00 %	● geeignet / suitable
Brial maxx	0,50 %	● geeignet / suitable
Cleanisept Wipes	–	● geeignet / suitable
Cosimed	2,00 %	● geeignet / suitable
Descogen Liquid	unverdünnt / pure	● geeignet / suitable
desderman pure	unverdünnt / pure	● geeignet / suitable
Franko-Cid N	1,00 %	● bedingt geeignet* / limited suitability*
Helipur H plus N	2,50 %	● geeignet / suitable
Hexaquant Plus lemon fresh	1,00 %	● nicht geeignet / not suitable
Incidin liquid	unverdünnt / pure	● geeignet / suitable
Kohrsolin FF	1,00 %	● geeignet / suitable
Lysoformin spezial	0,50 %	● geeignet / suitable
MAXX into Alk 2	0,50 %	● geeignet / suitable
Meliseptol	unverdünnt / pure	● geeignet / suitable
Meliseptol Foam pure	unverdünnt / pure	● geeignet / suitable
microzid AF liquid	unverdünnt / pure	● geeignet / suitable
Meliseptol rapid	unverdünnt / pure	● geeignet / suitable
Novadest fresh / Novadur	0,50 %	● geeignet / suitable
OPTISEPT	4,00 %	● geeignet / suitable
perform classic	unverdünnt / pure	● geeignet / suitable
terralin protect	2,00 %	● geeignet / suitable
Urinersatz	3,00 %	● geeignet / suitable

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* Limited suitability: Please test on a hidden section of the material before use. Do not let react for a long time and do not use for a longer period.

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Höpke Möbelstoff-Handels GmbH